IR-1S Armbandssändare
IR-1S Wrist transmitter
IR-1S Armbåndsender
IR-1S Armbandsender

Bruksanvisning / User manual / Brukerveiledning / Gebrauchsanweisung

Art. nr: 400500

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Introduction

Product description

The IR-1S Wrist transmitter is a single channel IR transmitter with internal setting to one of seven different channels. The IR-1S Wrist transmitter is a part of the “GewaLink-system” and therefore can be used together with all of these products.

Intended purpose


IR-1S Wrist transmitter is a simple remote control with only one button intended to support people who are unable to use a standard remote control due to motor, tactile or visual impairments. The device can help the user control electronic equipment at home, including lighting and door openers. The device is intended for indoor and outdoor use.

General information about IR-1S Wrist transmitter

On delivery the GewaLink channel 16 or 35 is factory programmed, this means that it is possible to use the transmitter as soon as a battery has been inserted.

The transmitter with a yellow function button is programmed with channel 16 and the transmitter with a black function button is programmed with channel 35.

Figure shows the various terms and functions which are described in the text.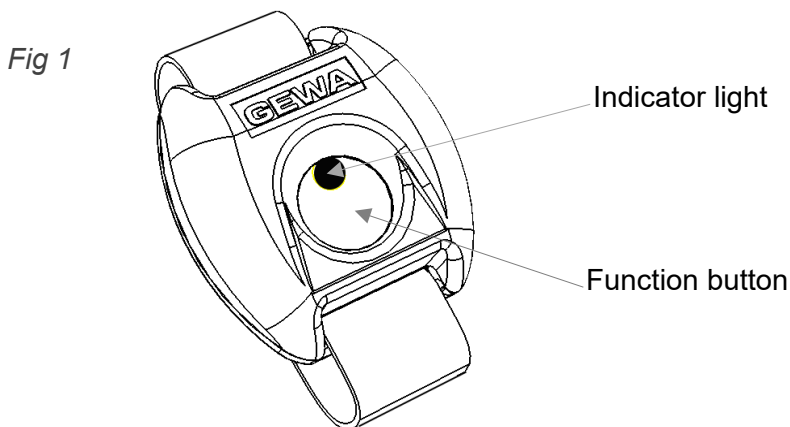


Inserting batteries

Unscrew the screws holding the bottom of the IR-1S Wrist transmitter using a Phillips screwdriver. Remove the bottom and insert three 1.5V silver oxide batteries type SR44, or 401930 Battery pack for Micro and Wrist transmitters, available from Abilia or Abilia dealer. Insert the batteries according to the guidelines at the bottom of the battery compartment. Pay attention to the polarity! It is important to use specified batteries for maximum transmission range.

Be sure the packing that seals the transmitter is correctly inserted.

Fig 2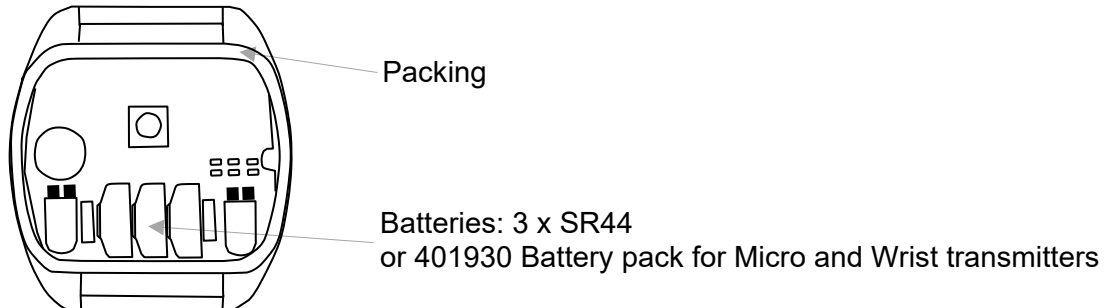


There will be a battery warning when 75% of the battery power has been used. The indicator lamp will flash with an interval of one second. The transmitter then has enough power to transmit for a while longer.

Channel setting

Channel settings are made by scraping or connecting joints between soldering pads on the circuit board. These spots are marked A, B and C in fig 3. The table shows which channels can be used and which joints should be scraped off or connected.

GewaLink-channel	Joints A	Joints B	Joints C
16	■	■	■
17	■	■	■
18	■	■	■
19	■	■	■
20	■	■	■
* 4096 code (random)	■	■	■
35	■	■	■

Fig 3

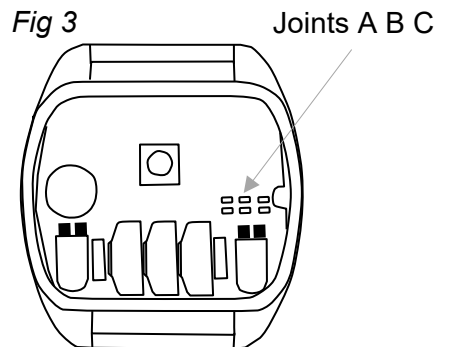* From serial number: 421012 channel 21 has been replaced with a random 4096 code.

Mounting IR-1S Wrist transmitter

⚠ WARNING: The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap.

IR-1S Wrist transmitter can be used either with the enclosed wrist strap or hanging on a strap around the neck.

Note! The neck strap is only included with the transmitter with a yellow function button.

Cleaning

Your IR-1S Wrist transmitter can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine, or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid). Wipe dry after cleaning.

Operating environment

The IR-1S Wrist transmitter is designed for use indoors and outdoors but should be protected against splashing liquid.

Technical data

Power consumption at rest:	Approx. 10 μ A
Power consumption at transmitting:	Approx. 10 mA (GewaLink channel)
Battery life:	Approx. 1 year (30 seconds transmitting per day with GewaLink channel)
Operating force button:	Approx. 170g
Range:	10-15m (GewaLink channels)
Measurements (L x W x H):	46 x 44 x 14 mm
Weight (incl. batteries):	25g
Operating temperature (excl. batteries):	-20°C to +45°C
Product life:	6 years

Accessories included with IR-1S Wrist transmitter

Art.no	Name	Description	Qty
890410	Neck strap	Allows IR-1S Wrist transmitter to be carried around the neck	1 pcs
419300	Battery pack for Micro and Wrist transmitters	Battery pack 1.5 V Silver oxide	1 pcs

Storage and disposal

Storage

Keep the product indoors at room temperature and away from moist.

Recycling Waste Electrical and Electronic Equipment

The IR-1S Wrist transmitter contains electronics components that must be recycled. When the product is no longer of use, hand it in to a recycling centre for waste electrical and electronic equipment.

Important information

Before you use the product, read and understand the contents of this user manual.

Safety definitions

⚠ WARNING: Risk of injury or death if the instructions are not followed.

⚠ CAUTION: Risk of damage to the product if the instructions are not followed.

NOTE: Information that is necessary in a given situation.

TIP: Additional information that can improve the use of the product.

Safety instructions before using IR-1S Wrist transmitter

⚠ WARNING: Read the following warning instructions before you use the product.

- The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Conformance with requirements for medical devices

IR-1S Wrist transmitter is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

Product liability

The product is developed and risk-assessed according to ISO 14971. The manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. Abilia AB does not take responsibility for any consequences following incorrect use or installation of the product. Any form of damage or tampering with the product invalidates the warranty and disclaims the manufacturer's liability. The estimated lifetime of the product applies when used in accordance with its intended use and maintenance instructions.

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Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Abilia UK Ltd, Unit 12 Buckingham Business Park, Swavesey,