IR-1SP Micro

Bruksanvisning / User manual / Brukerveiledning /
Manuel / Gebrauchsanweisung

Art. nr: 401104

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Introduction

Product description

The IR-1SP Micro is a programmable IR transmitter, which can learn one channel from another remote control for TV, DVD, stereo etc. The IR-1SP Micro is a part of the “GewaLink-system” and therefore can be used together with all of these products.

Intended purpose


IR-1SP Micro is a simple remote control with only one button intended to support people who are unable to use a standard remote control due to motor, tactile or visual impairments. The device can help the user control electronic equipment at home, including lighting and door openers. The device is intended for indoor and outdoor use.

General information about IR-1SP Micro

Before using your IR-1SP Micro there are certain things that you should know:

- On delivery the GewaLink channel 16 is factory programmed, meaning that it is possible to use the transmitter immediately after the batteries has been inserted. See section *Inserting Batteries*.
- For programming press the programming button **P** (using a pen tip or other narrow object) at the same time as you press the function button **O**.
- If you press the wrong button or make a mistake when programming you can always press the programming button **P**. It works as a *Reset*-button.
- It is important not to wait too long between the different steps when programming. After a certain time your IR-1SP Micro automatically leaves the programming mode to save power.

This picture shows the various terms and functions which are described in the text.

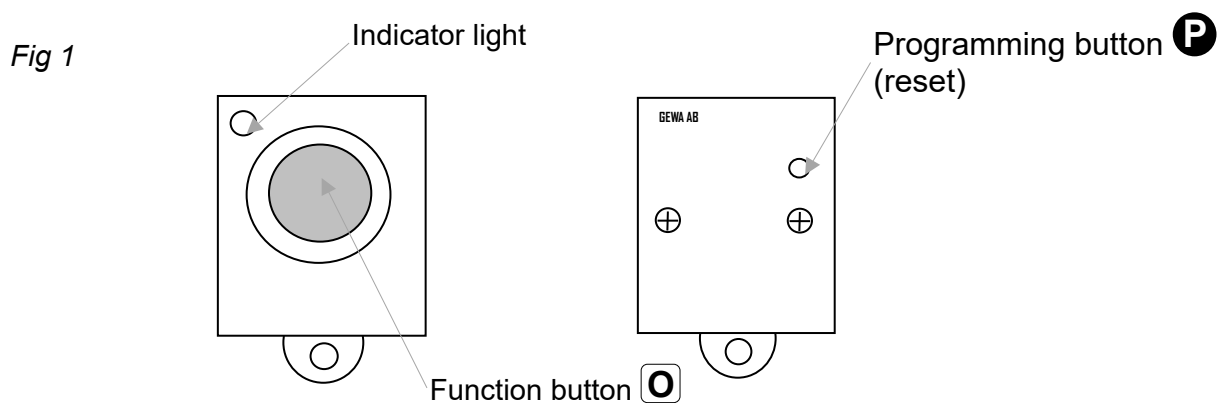The following symbols appear in the text throughout this manual:


- P** = programming button
- O** = function button on the IR-1SP Micro
- Y** = Any button on another transmitter to be programmed

Inserting Batteries

Unscrew the screws holding the bottom of the IR-1SP Micro using a Phillips screwdriver. Remove the bottom and insert three 1.5V silver oxide batteries type SR44, or 401930 Battery pack for Micro and Wrist transmitters, available from Abilia or Abilia dealer. Insert the batteries according to the guidelines at the bottom of the battery compartment. Pay attention to the polarity! It is important to use specified batteries for maximum transmission range.

When you have inserted the new batteries, press the programming button **P** on the underside.

Fig 2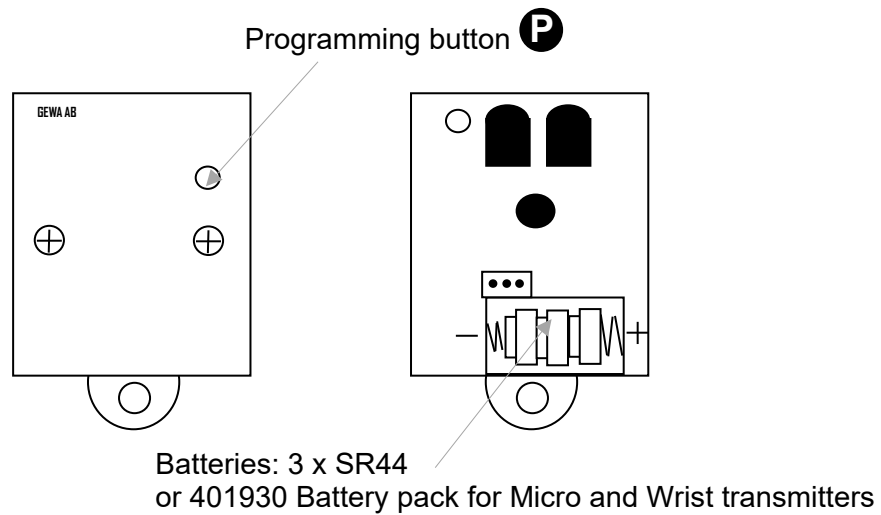

Recording IR Channel from another Transmitter


Make sure the batteries in both transmitters are fully charged. Do not use the transmitters in intense light (e.g., sunlight) when programming.

Place the two transmitters on a table with the original transmitter directed towards the front of your IR-1SP Micro, see picture.

1. Press the programming button **P** and the function button **O** at the same time.

To check that the transmitters are in the right position, keep any button pressed on the original transmitter and move the original transmitter to the left and right until the indicator light goes out. Then move the original transmitter backwards until the indicator light goes out (one metre for some transmitters). Place the original transmitter at the centre of these maximum range positions. Certain types of transmitters only emit a short flash each time the button is pressed. In these cases, press several times.

2. Press and release the function button **O** on the IR-1SP Micro. The indicator light now emits a steady red light.
3. Press and hold the button **Y** on the original transmitter that shall be recorded to IR-1SP Micro. Release the button when the indicator light goes out. It might take up to 10 seconds. If the indicator light turns red again then press the button again until it goes out.
4. Now test your recorded function on the device that is to be controlled.

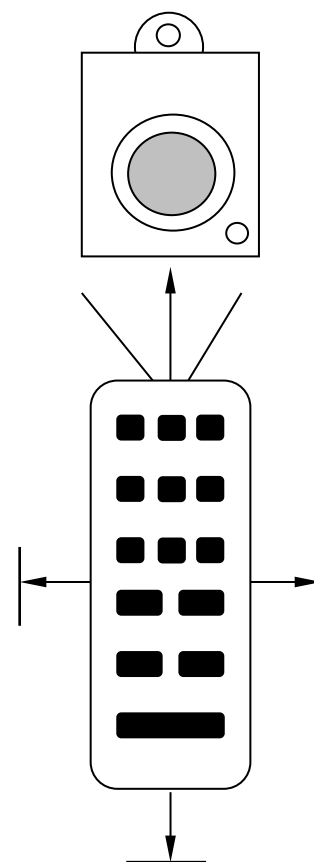Recording IR Channel

Press **P** + **O**, **O**, select button **Y**

TIP: IR-1SP Micro can record codes from most IR transmitters on the market with a few exceptions. You may have problems with a few or all of the functions for a certain IR transmitter. In these cases it is possible to use another recording method. Instead, do as follows in step 1; Press the programming button and the function button at the same time but keep them pressed for approximately 2 seconds. IR-1SP Micro is now in another recording mode. Otherwise use the same programming procedure.

Mounting IR-1SP Micro

⚠ WARNING: The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap.

The IR-1SP Micro can be used hanging on a strap around the neck. Thread the thin loop of the neck strap through the attachment loop on the transmitter. Adjust the length of the neck strap.

Cleaning

Your IR-1SP Micro can be cleaned with a damp cotton wool pad with surgical spirit, Chlorhexidine or ethyl alcohol.

It can also be cleaned with a damp cloth (weak solution of washing up liquid). Wipe dry after cleaning.

Operating Environment

The IR-1SP Micro is designed for use indoors and outdoors but should be protected against splashing liquid.

Technical data

Power consumption at rest:	Approx. 10 μ A
Power consumption at transmitting:	Approx. 10 mA (GewaLink channel)
Battery life:	Approx. 1 year (30 seconds transmitting per day with GewaLink channel)
Operating force button:	Approx. 170g
Range:	10-15m (GewaLink channels)
Measurements (L x W x H):	51 x 36 x 16 mm
Weight (incl. batteries):	30g
Operating temperature (excl. batteries):	-20°C to +45°C
Product life:	6 years

Accessories included with your Gewa IR-1SP Micro

Art.no	Name	Description	Qty
890410	Neck strap	Allows IR-1SP Micro to be carried around the neck	1 pcs
419300	Battery pack for Micro and Wrist transmitters	Battery pack 1,5 V Silver oxide	1 pcs

Storage and disposal

Storage

Keep the product indoors at room temperature and away from moist.

Recycling Waste Electrical and Electronic Equipment

The IR-1SP Micro contains electronics components that must be recycled. When the product is no longer of use, hand it in to a recycling centre for waste electrical and electronic equipment.

Important information

Before you use the product, read and understand the contents of this user manual.

Safety definitions

⚠ WARNING: Risk of injury or death if the instructions are not followed.

⚠ CAUTION: Risk of damage to the product if the instructions are not followed.

NOTE: Information that is necessary in a given situation.

TIP: Additional information that can improve the use of the product.

Safety instructions before using IR-1SP Micro

⚠ WARNING: Read the following warning instructions before you use the product.

- The string lock is a safety feature to prevent strangling. Make sure that the ends of the neck strap can be pulled out eg. if the transmitter gets caught on something. Never make knots in the neck strap.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Conformance with requirements for medical devices

IR-1SP Micro is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

Product liability

The product is developed and risk-assessed according to ISO 14971. The manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. Abilia AB does not take responsibility for any consequences following incorrect use or installation of the product. Any form of damage or tampering with the product invalidates the warranty and disclaims the manufacturer's liability. The estimated lifetime of the product applies when used in accordance with its intended use and maintenance instructions.

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Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Abilia UK Ltd, Unit 12 Buckingham Business Park, Swavesey,